

FOX MOBILE Inhalation Set



Instructions for Use



Warning:

The FOX MOBILE Inhalation Set is an accessory to the FOX MOBILE Inhalation System. Read the entire instructions for use of the FOX MOBILE Inhalation System before operating the device. Correct and safe operation can only be achieved if instructions for use of the FOX MOBILE Inhalation System are followed.

Please use the following email address if you have technical problems or questions concerning the FOX MOBILE:

Email: fox.mobile@vectura.com



Vectura Group Ltd
One Prospect West
Chippenham, Wiltshire, SN14 6FH
United Kingdom
www.vectura.com

CE 0123



Vectura Ireland Limited
Block 2
Newtown Business and
Enterprise Centre
Newtownmountkennedy
Co. Wicklow, A63 XV26
Ireland

1. Safety Instructions

Accurate knowledge of and compliance with these operating instructions is required before the FOX MOBILE is used. The operator of the FOX MOBILE is liable for its safe functioning in all cases when it is handled incorrectly. It must be repaired only by Vectura Group Ltd or by a service depot expressly authorised by Vectura Group Ltd. Vectura Group Ltd is not liable for any damage or malfunction resulting from incorrect use or from use for which the device is not intended.



Warning:

- The FOX MOBILE is designed for single patient use. Do not share your FOX MOBILE with other people.
- Devices must be cleaned and disinfected according to the Instructions for Use of the FOX MOBILE Inhalation System.
- The Inhalation Set may only be used for a maximum of 14 days after 1st use, or 42 treatments, whichever comes first.
- Do not use the device if it is damaged.

Instructions for Use

2. Package Content

- 1 x Nebuliser Reservoir
- 1 x Nebuliser Cap
- 1 x Mouthpiece
- 1 x Instructions for use (this document)

3. Intended Use, General Instruction, Use of the device

The FOX MOBILE Inhalation Set is an accessory to the FOX MOBILE Inhalation System. The FOX MOBILE Inhalation Set can only be used in combination with the FOX MOBILE Inhalation System. For information regarding Intended Use, General Instructions and the Use of the device refer to instructions for use of your FOX MOBILE Inhalation System.











4. Storage and Transport

No specific storage and transport conditions are defined for the FOX MOBILE Inhalation Set.

5. Service Life

Replace the Mouthpiece, Nebuliser Reservoir and Nebuliser Cap at the end of service life. The components delivered with the FOX MOBILE Inhalation Set may be used up to a maximum of 14 days after 1st use, or 42 treatments, whichever comes first.

6. Explanation of Symbols used

SYMBOL	INDICATION
	This device complies with the requirements of the Medical Devices Directive (93/42/EEC)
	Follow the Instructions For Use
	Manufacturer
	Reference number (article number)
	Serial number
	Indicates the authorised representative in the European Community
	Importer
	Medical device
	Unique device identifier
	Use-by date

Information as of: March 2025

DAW-00044 Rev.06

Item Code 500044R6

Instructions for Use

7. Disposal

Used and cleaned parts of the Inhalation Set can be disposed of in the normal household waste.